

New FDA-approved *Natrelle* INSPIRA™

Designed for increased *Fullness*
and a soft feel.



And, with 100 more sizes, *Natrelle*® Gel offers the most options,
so you can give patients a customized fit.¹⁻⁵

***Natrelle* INSPIRA™ Breast Implants Important Information**

INDICATIONS

Natrelle INSPIRA™ Breast Implants are indicated for women for the following:

- **Breast augmentation for women at least 22 years old for silicone-filled implants.**

Breast augmentation includes primary breast augmentation to increase breast size, as well as revision surgery to correct or improve the result of a primary breast augmentation surgery.

Please see additional *Natrelle* INSPIRA™ Breast Implants Indications and Important Safety Information on following pages.

Natrelle INSPIRA™
ROUND GEL

Increased *Fullness* with more options.

Natrelle INSPIRA™ Round Gel Breast Implants are for patients who want increased fullness with a soft feel.

More options for a customized fit:

- 100 smooth round sizes
- 100 matching re-sterilizable sizers
- 5 projection profiles
 - Extra-Full
 - Full
 - Moderate
 - Low-Plus
 - Low



- Consistent diameter sizing available in 0.5-cm increments along with 0.25-cm increments available for commonly used sizes

Think *Natrelle INSPIRA™* for women who:

- Want augmentation
- Are post partum
- Are due for a revision
- Need reconstruction

Natrelle INSPIRA™ Breast Implants Important Information (continued)

INDICATIONS (continued)

- **Breast reconstruction.** Breast reconstruction includes primary reconstruction to replace breast tissue that has been removed due to cancer or trauma or that has failed to develop properly due to a severe breast abnormality. Breast reconstruction also includes revision surgery to correct or improve the result of a primary breast reconstruction surgery.

IMPORTANT SAFETY INFORMATION

CONTRAINDICATIONS

- Breast implant surgery should not be performed in:
- Women with active infection anywhere in their body.
 - Women with existing cancer or pre-cancer of their breast who have not received adequate treatment for those conditions.
 - Women who are currently pregnant or nursing.

Before-and-After Photos

Preoperation



6 weeks post operation



Primary Augmentation
 Implant selected: *Natrelle INSPIRA™* Style SRF-415
 Implant dimensions: 415 cc volume, 12.50 cm diameter, 5.1 cm projection
 Photos provided by: Dr Mitchell Brown.
Individual results may vary.

Preoperation



1 year post operation



Primary Augmentation
 Implant selected: *Natrelle INSPIRA™* Style SRF-385
 Implant dimensions: 385 cc volume, 12.25 cm diameter, 5.0 cm projection
 Photos provided by: Dr Paul G.R. Whidden.
Individual results may vary.

WARNINGS

AVOID DAMAGE DURING SURGERY

- Care should be taken to avoid the use of excessive force and to minimize handling of the implant during surgical insertion.
- Care should be taken when using surgical instruments in proximity with the breast implant, including scalpel, sutures, and dissection instrumentation.

Please see additional *Natrelle INSPIRA™* Breast Implants Important Safety Information on following pages.



Natrelle INSPIRA™. More sizes. More options.

Within the 11.5-cm to 13.5-cm diameter range, *Natrelle INSPIRA™* has:

46% more
smooth round implant options
than *Natrelle®* Classic Round Gel Breast Implants^{1,2}

64% more
smooth round implant options than *Mentor®*^{1,3}

64% more
smooth round implant options than *Sientra®*^{1,5}

Natrelle INSPIRA™ Style SRL Smooth, Responsive Gel, Low Profile

Catalog Number	Volume (cc)	Diameter (cm)	Projection (cm)	Re-sterilizable Sizer
SRL-110	110	10.00	2.0	MSZ-L110
SRL-125	125	10.25	2.0	MSZ-L125
SRL-140	140	10.50	2.1	MSZ-L140
SRL-170	170	11.00	2.2	MSZ-L170
SRL-200	200	11.50	2.3	MSZ-L200
SRL-230	230	12.00	2.4	MSZ-L230
SRL-260	260	12.50	2.5	MSZ-L260
SRL-290	290	13.00	2.6	MSZ-L290
SRL-320	320	13.50	2.7	MSZ-L320
SRL-350	350	14.00	2.8	MSZ-L350
SRL-380	380	14.50	2.9	MSZ-L380
SRL-410	410	15.00	3.0	MSZ-L410
SRL-460	460	15.50	3.1	MSZ-L460
SRL-510	510	16.00	3.2	MSZ-L510
SRL-560	560	16.50	3.3	MSZ-L560
SRL-610	610	17.00	3.4	MSZ-L610

Natrelle INSPIRA™ Style SRLP Smooth, Responsive Gel, Low-Plus Profile

Catalog Number	Volume (cc)	Diameter (cm)	Projection (cm)	Re-sterilizable Sizer
SRLP-125	125	9.50	2.5	MSZ-LP125
SRLP-145	145	10.00	2.6	MSZ-LP145
SRLP-165	165	10.50	2.7	MSZ-LP165
SRLP-190	190	11.00	2.8	MSZ-LP190
SRLP-205	205	11.25	2.9	MSZ-LP205
SRLP-220	220	11.50	3.0	MSZ-LP220
SRLP-235	235	11.75	3.1	MSZ-LP235
SRLP-250	250	12.00	3.1	MSZ-LP250
SRLP-265	265	12.25	3.2	MSZ-LP265
SRLP-280	280	12.50	3.3	MSZ-LP280
SRLP-300	300	12.75	3.3	MSZ-LP300
SRLP-320	320	13.00	3.4	MSZ-LP320
SRLP-340	340	13.25	3.5	MSZ-LP340
SRLP-360	360	13.50	3.5	MSZ-LP360
SRLP-400	400	14.00	3.6	MSZ-LP400
SRLP-440	440	14.50	3.8	MSZ-LP440
SRLP-490	490	15.00	3.9	MSZ-LP490
SRLP-540	540	15.50	4.0	MSZ-LP540
SRLP-590	590	16.00	4.2	MSZ-LP590
SRLP-640	640	16.50	4.3	MSZ-LP640

Natrelle INSPIRA™ Breast Implants IMPORTANT SAFETY INFORMATION (continued)

WARNINGS (continued)

- Do not treat capsular contracture by closed capsulotomy or forceful external compression, which will likely result in implant damage, rupture, folds, and/or hematoma.
- Use care in subsequent procedures such as open capsulotomy, breast pocket revision, hematoma/seroma aspiration, and biopsy/lumpectomy to avoid damage to the implant.
- Do not contact the implant with disposable, capacitor-type cautery devices.
- Do not alter the implants or attempt to repair or insert a damaged prosthesis.
- Do not immerse the implant in povidone-iodine solution. If povidone-iodine is used in the pocket, ensure that it is rinsed thoroughly so no residual solution remains in the pocket.

- Do not reuse or resterilize any product that has been previously implanted. Breast implants are intended for single use only.
- Do not place more than one implant per breast pocket.
- Do not use the periumbilical approach to place the implant.
- Do not use microwave diathermy in patients with breast implants. Microwave diathermy has been reported to cause tissue necrosis, skin erosion, and implant extrusion.

Please see additional *Natrelle INSPIRA™* Breast Implants Important Safety Information on back page.

Natrelle INSPIRA™ Style SRM Smooth, Responsive Gel, Moderate Profile

Catalog Number	Volume (cc)	Diameter (cm)	Projection (cm)	Re-sterilizable Sizer
SRM-140	140	9.50	3.0	MSZ-M140
SRM-175	175	10.00	3.3	MSZ-M175
SRM-195	195	10.25	3.4	MSZ-M195
SRM-210	210	10.50	3.5	MSZ-M210
SRM-240	240	11.00	3.7	MSZ-M240
SRM-255	255	11.25	3.8	MSZ-M255
SRM-275	275	11.50	3.9	MSZ-M275
SRM-295	295	11.75	3.9	MSZ-M295
SRM-310	310	12.00	4.0	MSZ-M310
SRM-330	330	12.25	4.1	MSZ-M330
SRM-345	345	12.50	4.2	MSZ-M345
SRM-360	360	12.75	4.2	MSZ-M360
SRM-375	375	13.00	4.3	MSZ-M375
SRM-405	405	13.25	4.4	MSZ-M405
SRM-445	445	13.50	4.5	MSZ-M445
SRM-485	485	14.00	4.6	MSZ-M485
SRM-520	520	14.50	4.8	MSZ-M520
SRM-560	560	14.75	4.9	MSZ-M560
SRM-600	600	15.00	4.9	MSZ-M600
SRM-640	640	15.50	5.0	MSZ-M640
SRM-685	685	16.00	5.2	MSZ-M685
SRM-755	755	16.50	5.0	MSZ-M755

Natrelle INSPIRA™ Style SRF Smooth, Responsive Gel, Full Profile

Catalog Number	Volume (cc)	Diameter (cm)	Projection (cm)	Re-sterilizable Sizer
SRF-180	180	9.50	4.0	MSZ-F180
SRF-200	200	9.75	4.1	MSZ-F200
SRF-220	220	10.00	4.2	MSZ-F220
SRF-240	240	10.50	4.3	MSZ-F240
SRF-265	265	11.00	4.4	MSZ-F265
SRF-295	295	11.25	4.5	MSZ-F295
SRF-325	325	11.40	4.6	MSZ-F325
SRF-335	335	11.50	4.7	MSZ-F335
SRF-345	345	11.75	4.8	MSZ-F345
SRF-365	365	12.00	4.9	MSZ-F365
SRF-385	385	12.25	5.0	MSZ-F385
SRF-415	415	12.50	5.1	MSZ-F415
SRF-450	450	12.75	5.3	MSZ-F450
SRF-485	485	13.00	5.4	MSZ-F485
SRF-520	520	13.25	5.5	MSZ-F520
SRF-560	560	13.50	5.7	MSZ-F560
SRF-605	605	14.00	5.8	MSZ-F605
SRF-650	650	14.50	5.9	MSZ-F650
SRF-695	695	14.75	6.0	MSZ-F695
SRF-745	745	15.00	6.1	MSZ-F745
SRF-770	770	15.50	6.0	MSZ-F770

Natrelle INSPIRA™ Style SRX Smooth, Responsive Gel, Extra-Full Profile

Catalog Number	Volume (cc)	Diameter (cm)	Projection (cm)	Re-sterilizable Sizer
SRX-200	200	9.50	4.2	MSZ-X205
SRX-225	225	10.00	4.5	MSZ-X225
SRX-255	255	10.25	4.8	MSZ-X255
SRX-285	285	10.50	5.0	MSZ-X285
SRX-310	310	10.75	5.2	MSZ-X310
SRX-340	340	11.00	5.4	MSZ-X340
SRX-375	375	11.25	5.6	MSZ-X375
SRX-400	400	11.50	5.8	MSZ-X400
SRX-420	420	11.75	5.9	MSZ-X420
SRX-445	445	12.00	6.0	MSZ-X445
SRX-470	470	12.25	6.1	MSZ-X470
SRX-495	495	12.50	6.1	MSZ-X495
SRX-525	525	12.75	6.2	MSZ-X525
SRX-545	545	13.00	6.2	MSZ-X545
SRX-560	560	13.25	6.3	MSZ-X560
SRX-580	580	13.50	6.4	MSZ-X580
SRX-615	615	13.75	6.4	MSZ-X615
SRX-650	650	14.00	6.5	MSZ-X650
SRX-700	700	14.25	6.6	MSZ-X700
SRX-750	750	14.50	6.7	MSZ-X750
SRX-800	800	15.50	6.7	MSZ-X800



Natrelle **INSPIRA™**
ROUND GEL

Increased *Fulness*, with more options.

Natrelle INSPIRA™ Breast Implants IMPORTANT SAFETY INFORMATION (continued)

PRECAUTIONS

Safety and effectiveness have not been established in patients with the following:

- Autoimmune diseases (e.g., lupus and scleroderma).
- A compromised immune system (for example, currently receiving immunosuppressive therapy).
- Planned chemotherapy following breast implant placement.
- Planned radiation therapy to the breast following breast implant placement.
- Conditions or medications that interfere with wound healing and blood clotting.
- Reduced blood supply to breast tissue.
- Clinical diagnosis of depression or other mental health disorders, including body dysmorphic disorder and eating disorders. Please discuss any history of mental health disorders prior to surgery. Patients with a diagnosis of depression, or other mental health disorders, should wait until resolution or stabilization of these conditions prior to undergoing breast implantation surgery.

ADVERSE EVENTS

Key adverse events are reoperation, implant removal with or without replacement, implant rupture with silicone-filled implants, and capsular contracture Baker Grade III/IV.

Other potential adverse events that may occur with breast implant surgery include: asymmetry, breast pain, breast/skin sensation changes, capsular calcification, delayed wound healing, hematoma, hypertrophic scarring/scarring, implant extrusion, implant malposition, implant palpability/visibility, infection, nipple complications, redness, seroma, swelling, tissue/skin necrosis, wrinkling/rippling.

For more information see the full Directions for Use at www.allergan.com/labeling/usa.htm or call the Allergan Product Support line at 1-800-433-8871.

To report a problem with *Natrelle* INSPIRA™, please call Allergan Product Surveillance at 1-800-624-4261.

Natrelle INSPIRA™ Breast Implants are available by prescription only.

Intraoperative Breast Implant Sizers Important Information INDICATIONS

The *Natrelle*® Silicone Sizer and the Allergan Saline Sizer are indicated for single use only for temporary intra-operative insertion in the surgical pocket to evaluate and assist in determining the final breast implant size/volume. The *Natrelle*® Re-sterilizable (410 and Round) Silicone Breast Implant Sizer is used during breast augmentation or reconstruction procedures to assist the surgeon in determining the appropriate size of a breast implant to use.

IMPORTANT SAFETY INFORMATION CONTRAINDICATIONS

All sizers are contraindicated for use as long-term breast implants or tissue expanders. The *Natrelle*® Silicone Sizer and the Allergan Saline Sizer are contraindicated for multiple patient use or multiple sterilizations.

WARNINGS

Sizers are designed for temporary intra-operative use only and are NOT long-term implants. DO NOT alter, insert or attempt to repair a damaged sizer. DO NOT reuse the *Natrelle*® Silicone Sizer or the Allergan Saline Sizer, which are for single use only. The Silicone Sizers may rupture and release silicone gel. Infection, necrosis, hematoma/seroma and pain may occur following any type of surgery. Minute quantities of silicone gel may diffuse through the elastomer envelope.

PRECAUTIONS

The surgeon must carefully evaluate patient suitability and be knowledgeable about the use of this device. DO NOT expose the sizer to contaminants. Avoid damaging the sizer with surgical instruments (e.g. sharp, blunt or cautery devices). DO NOT attempt to repair damaged products. DO NOT damage the sizer by overhandling, manipulation, or excessive force. Maintain a sterile back-up sizer during surgery.

ADVERSE EVENTS

Adverse events and/or complications may include sepsis, hemorrhage, thrombosis, bleeding, and/or infection.

To report a problem, please call Allergan Product Surveillance at 1-800-624-4261.

For more information, please visit www.allergan.com/labeling/usa.htm or call the Allergan Product Support line at 1-800-433-8871.

Intraoperative Breast Implant Sizers are available by prescription only.

References: 1. Data on file, Allergan, Inc., February 2015; *Natrelle* INSPIRA™ Smooth and Textured Implants. 2. The *Natrelle*® Product Catalog. Irvine, CA: Allergan, Inc.; 2014. 3. Our products: breast implants. Mentor Worldwide LLC website. <http://www.mentorwwllc.com/global-us/Breast.aspx>. Accessed March 18, 2015. 4. *Sientra*® Breast Implants and Tissue Expanders Product Catalog. Santa Barbara, CA: Sientra, Inc.; 2012. http://www.sientra.com/Content/pdfs/Product_Catalog_Breast_Implants_and_Tissue_Expanders.pdf. Accessed March 18, 2015. 5. *Sientra*® HSC Smooth Round Implants Product Insert. Sientra website. http://sientra.com/Content/pdfs/MDC-0148_R1_HSC_Smooth_Round_Product_Insert.pdf. Accessed March 4, 2015.